Rotaid AED Cabinets

SOLID PLUS HEAT

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www.rotaid.com | info@rotaid.com | +31 – 478 769 052

User guide

On the Rotaid AED Cabinets website, www.rotaid.com/support, further information and video material can be found.

LOCATION AED CABINET

Make sure there are no irregularities present at the desired location. Please take the fire safety and evacuation regulations into account. If mounted on a stone or concrete wall, please use the wall anchors prior to fixation.

To ensure the defibrillator stays within its operational temperature range during the whole year, the Rotaid should be located in the shade at all times.

OPENING CABINET

All our cabinets only require a short twist to the left to open. The whole cover comes off to provide full access to the defibrillator. The cover is fixated to the backplate via a belt and does not require your attention after opening.

ALARM

The alarm is triggered when the cabinet is opened and will sound continuously for 15 minutes. After 15 minutes it will beep every 5 minutes. On the bottom of the alarm unit, a small red button can be found. When the cabinet is in alarm, you push this button to temporarily turn off the alarm. When the cover is placed back onto the cabinet, a dual tone signal is heard. This signal sounds to confirm that the alarm is functioning again.

Two AA-batteries power the alarm system for at least 24 months and can be found in the fastening materials package.

At the bottom there is also a white switch to select the desired loudness (84 dBa and 98 dBa).

PLACEMENT AED

If placed in an outdoor environment, always make sure the AED comes in the official manufacturer's bag for outdoor use!

For the Plus, Solid Plus and Solid Plus Heat, use the two long bolts and pre-drilled holes on the top of the backplate to find the ideal position to hang the defibrillator via its handle or carrying case.

When placing the AED in the cabinet, make sure the backside of the bag is as close to the heating system as possible.

CLOSING CABINET

To close the cabinet, place the cover about 15 centimetres to the left (counter clockwise) and then twist to the right until the cover cannot move any further. Please also make sure to enclose the steel hook in the cover

when placing the cover when using a Plus, Solid Plus or Solid Plus Heat LED. In the video section on this page a more detailed video can be found regarding the closing of the cabinet.

Maintenance advice

COVER

The cover needs no maintenance. For a high-quality appearance, you can clean the cover from time to time with a damp cloth to remove dirt and any residual sand.

BACKPLATE

The back plate requires no maintenance. In the backplate, a rubber ring/seal is mounted to ensure your cabinet is watertight. To make sure this rubber seal remains flexible, please lubricate the seal at least twice a year with a greasy liquid or spray, please do this once before the summer and once before the winter.

HEATING & LED LIGHTS

The LEDs burn continuously. This makes your AED visible during the dark hours.

The heater works fully automated and will switch on during cold temperatures thanks to the built-in thermostat and controller. Please make sure the socket that powers the heating device will be marked to prevent loss of power.

In case of a malfunction, the LEDs will blink. Your heating device will not work anymore. If so, contact your official Rotaid AED Cabinets distributor for further assistance.

Alarm System

For optimal performance, replace both AA batteries once every 24 months. Only use Alkaline batteries for optimal performance and life span.

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